


Query: Entering History, Image Count & Patient Location

Tech Support Phone: 866-717-3627 Dial "5"



Go to: <https://newprod.imagingoncall.com/amicas-viewer>

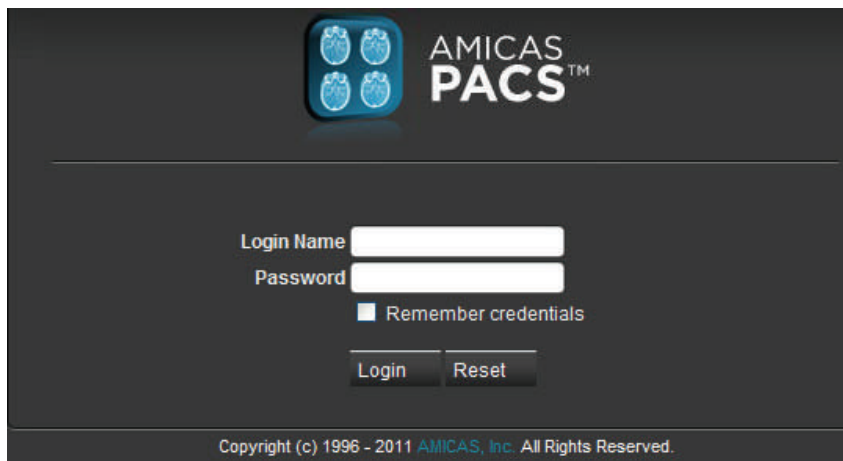

AMICAS PACS™

Login Name


Password






Remember credentials

Login Reset

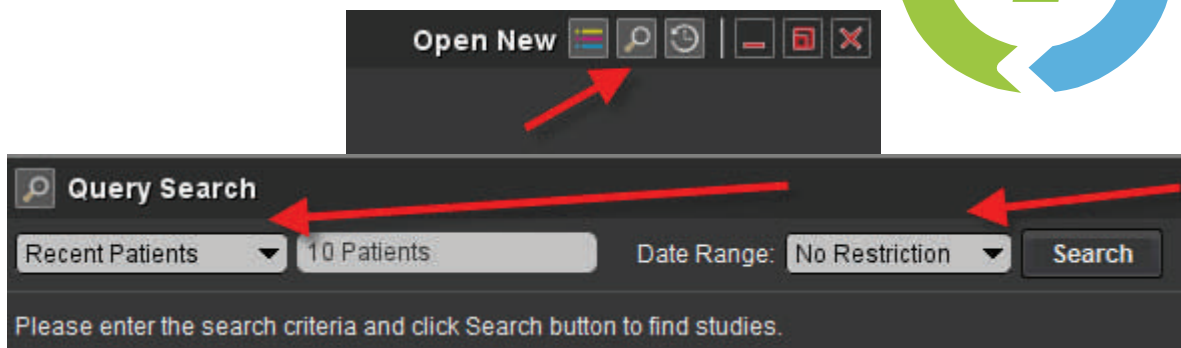Copyright (c) 1996 - 2011 AMICAS, Inc. All Rights Reserved.

Click on the query icon and enter search parameters



Open New     

Query Search

Recent Patients Date Range: Search

Please enter the search criteria and click Search button to find studies.




After you enter your search criteria your patient or patients will appear on worklist

| Actions | Diagnostic | Patient Name | MRN | Accession | Modality | Date of Birth | Procedure |
|---------------|------------|--------------|-----|-----------|----------|---------------|-----------|
| June 30, 2011 | | | | | | | |

Entering History, Image Ct & Pt location



When you see your patient on the worklist, click on the “tech panel icon” for your patient

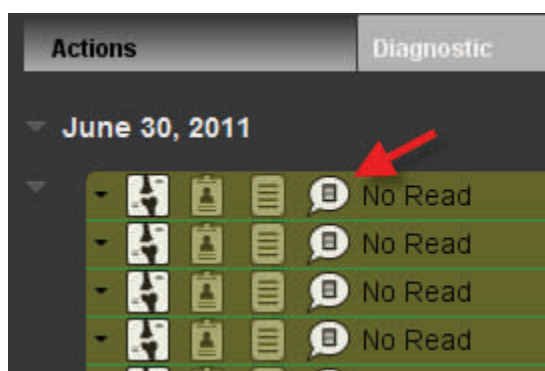

Tech “work panel” pops open and enter in the “comments” section

*Reason For Exam

*# of images or sequences

*Patient location

Then hit “save” to publish your comments

A screenshot of a software form titled 'Exam Information'. On the left, there are fields for 'Physician:' (Not Available), 'Name:' (Not Available), 'Phone:', and 'Fax:'. On the right, there are 'View:' options (Communication Log | User Access | Manual Validation), a 'Status:' dropdown (Please Select), and a 'Comments:' text input field. Below the 'Comments' field is a 'Commenter:' dropdown (toshca) and a 'Publish:' checkbox (checked). At the bottom right are 'Save' and 'Clear' buttons. A red arrow points to the 'Comments' text input field.