

# RTWL: Entering History, Image Count & Patient Location



Tech Support Phone: 866-717-3627 Dial "5"



Go to: <https://newprod.imagingoncall.com/amicas-viewer>

A screenshot of the AMICAS PACS login interface. At the top left is the AMICAS PACS logo, which consists of four brain icons in a 2x2 grid. Below the logo are two input fields for "Login Name" and "Password". There is a checkbox labeled "Remember credentials" and two buttons, "Login" and "Reset". At the bottom, there is a copyright notice: "Copyright (c) 1996 - 2011 AMICAS, Inc. All Rights Reserved."

Click the drop down below Real Time Worklist and select your site

A screenshot of a "RealTime Worklist" dropdown menu. The menu is open, showing a "Please Select" option. A red arrow points to the dropdown arrow, and a mouse cursor is visible over the menu.


After you select your site all recent patients will appear in Worklist

Actions	Diagnostic	Patient Name	MRN	Accession	Modality	Date of Birth	Procedure
June 30, 2011							

# Entering History, Image Ct & Pt location



When the worklist appears, click on the “tech panel icon” for your patient

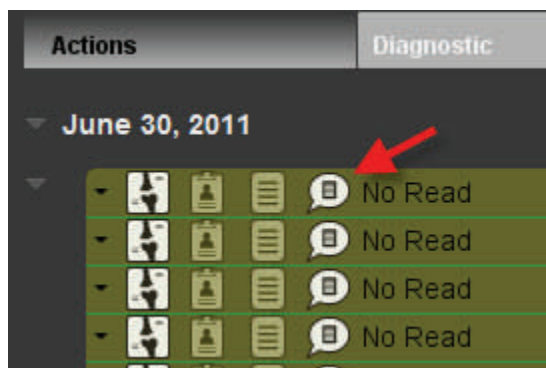


Tech “work panel” pops open and enter in the “comments” section

\*Reason For Exam

\*# of images or sequences

\*Patient location

Then hit “save” to publish your comments

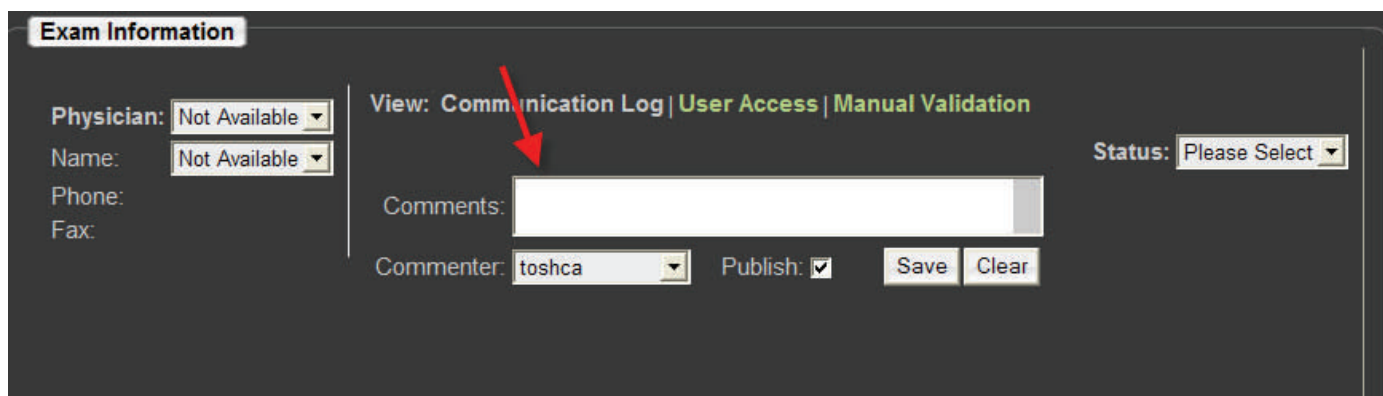

**Exam Information**

Physician:

Name:

Phone:

Fax:

View: [Communication Log](#) | [User Access](#) | [Manual Validation](#)

Status:

Comments:

Commenter:  Publish:  Save Clear